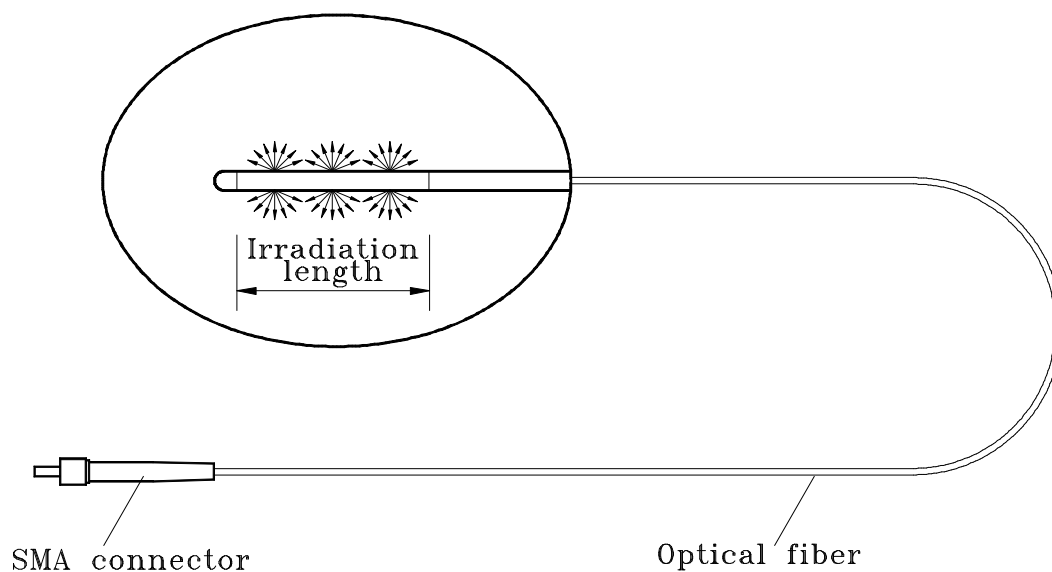


Ch. des Larges-Pièces B, CH-1024 Ecublens

phone: +41 / 21 - 697 - 07 75
fax: +41 / 21 - 697 - 07 79
e-mail: info@medlight.com
web site: www.medlight.com

Cylindrical light diffuser Model RD

CE 1253

DESCRIPTION

The cylindrical light diffuser is a radially emitting fiber, which can be used for intraluminal photodynamic therapy (PDT). The cylindrical diffuser is thin and very flexible, making it the optimal choice for endoscopic use.

It produces a radial light pattern, which is homogeneous all along the diffuser tip.

Standard tip lengths are 10, 20, 25, 30, 40, 50 and 70 mm. Other lengths can be manufactured on request. The cylindrical light diffuser can be supplied with radio marker bands (option).

The diffuser is supplied sterile in peel-open package and is intended for single use only.

The RD diffuser must only be used with approved Photodynamic Therapy protocol.

TECHNICAL DATA

MECHANICAL DIMENSIONS	RD Model
OVERALL DIAMETER	0.98 mm
OVERALL LENGTH	2.5 m
OPTICAL CHARACTERISTICS	
TRANSMISSION @ 630 / 652 / 690 nm (*)	70 / 85 / 75 %
ILLUMINATION LENGTH	
STANDARD LENGHTS [mm]:	10, 20, 25, 30, 40, 50, 70
OTHER LENGHTS ON REQUEST	
UNIFORMITY	± 15%
MAXIMUM POWER DENSITY (in air)	0.5 W/cm (cw)
ABSOLUTE MAXIMUM INPUT POWER	2.0 W (cw)
OPTICAL FIBER	
FIBER MATERIAL	PLASTIC
CORE DIAMETER	500 µm
NUMERICAL APERTURE	0.48
MINIMUM BENDING RADIUS	10 mm
FIBER CONNECTOR	SMA 905
CONDITIONING	
PACKAGING	INDIVIDUAL POUCH
STERILIZATION	STERILE / EtO
USEFUL LIFE	
REUSE	DISPOSABLE
SHELF LIFE	2 YEARS
MISCELLANEOUS	
RADIOMARKERS	OPTIONAL

(*) Transmission is defined in comparison with a 5 meters / 600 microns / NA 0.37 silica bare fiber.

TYPICAL LIGHT INTENSITY PROFILE

(measured in air, in contact with the diffuser's surface)

